

ECHELON CIRCULAR™ Powered Stapler

How to release the anastomosis and remove from patient


For product details, including a complete list of warnings and precautions, please reference the Instructions for Use.



Releasing the anastomosis:

After device has been fired, open device by turning Adjusting Knob counterclockwise for **2 complete revolutions**.

Note: This step is different from other Ethicon circular staplers.

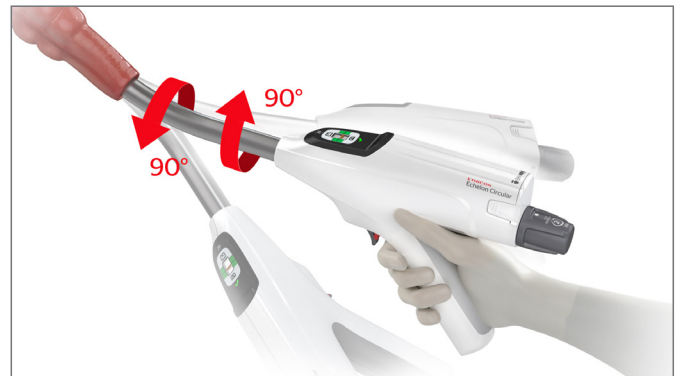

Removing from patient:

1. Rotate device 90° in both directions taking care to minimize movement of the distal tip. This ensures the tissue is released.
2. Gently pull out device while simultaneously rotating device back 90°.
3. Repeat as required until device slides out.